


















Drilling Sequence Overview

Standard protocol

For more information read the Surgical Procedures Manual

Rectangles, indicate drills that are included in the Single Patient Drill Kit

 <p>OsseoSpeed™ 3.5 S</p>				 <p>OsseoSpeed™ 4.0 S</p>						
	Guide Drill	Twist Drill 2.0	Twist Drill 3.2		Guide Drill	Twist Drill 2.0	Twist Drill 3.2	Twist Drill 3.7		
 <p>OsseoSpeed™ 4.5</p>					 <p>OsseoSpeed™ 5.0</p>					
	Guide Drill	Twist Drill 2.0	Twist Drill 3.2	Conical Drill 4.5		Guide Drill	Twist Drill 2.0	Twist Drill 3.2	Twist Drill 3.7	Conical Drill 5.0
 <p>OsseoSpeed™ 5.0 S</p>										
	Guide Drill	Twist Drill 2.0	Twist Drill 3.2	Twist Drill 3.7	Twist Drill 4.2	Twist Drill 4.7				

Single Patient Drill Kits

Stainless steel
Sterile
Laser etched depth indications.

For simplified ordering and inventory, initial drills required for all cases are conveniently packaged together.

Guide Drill
SP Twist Drill, 2.0 mm
SP Twist Drill, 3.2 mm



Short, REF 22898



Long, REF 22899

Single Patient Drills

	REF	Short	REF	Long
SP Twist Drill, 3.35 mm	22908		22909	
SP Twist Drill, 3.7 mm	22910		22911	
SP Twist Drill, 3.85 mm	22912		22913	
SP Twist Drill, 4.2 mm	22957		22958	
SP Twist Drill, 4.7 mm	22959		22960	
SP Twist Drill, 4.85 mm	22961		22962	
SP Pilot Drill 2.0/3.2 mm	22904		22905	
SP Pilot Drill, 3.2/3.7 mm	22906		22907	
SP Pilot Drill, 3.7/4.2 mm	22963			
SP Cortical Drill, 3.5 mm	22914		22915	
SP Cortical Drill, 4.0 mm	22916		22917	
SP Cortical Drill, 5.0 mm	22964			
SP Conical Drill, 4.5 mm	22921		22922	
SP Conical Drill, 5.0 mm	22923		22924	



Single Patient Drills are disposable and packaged sterile, providing unmatched cutting efficiency for optimal treatment outcomes.

